



Eaton's face shield assembly and care instructions

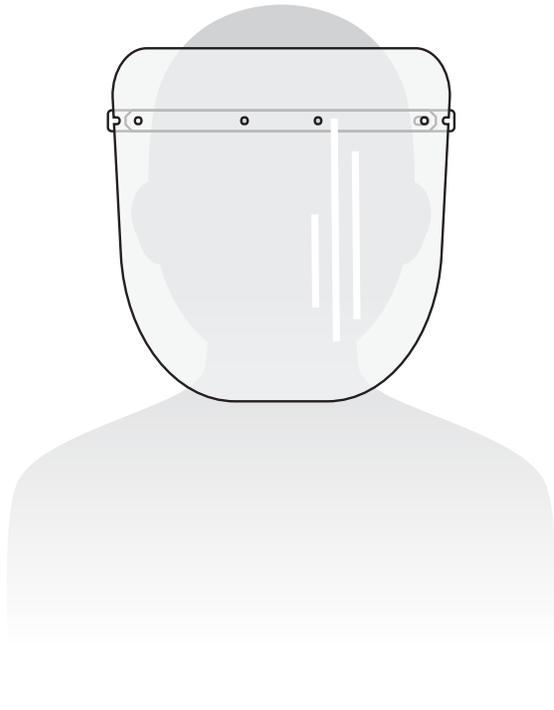

Parts

Face shield

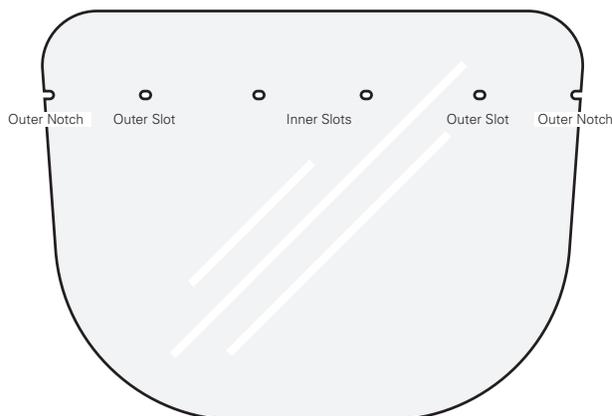

WARNING

READ AND UNDERSTAND THIS WARNING BEFORE USE.

THIS FACE SHIELD MAY HELP REDUCE EXPOSURE DURING SURGERIES, PROCEDURES, OR OTHER CARE ACTIVITIES WHERE SPLASHES AND SPRAYS ARE ANTICIPATED, OR DURING THE COURSE OF AEROSOL GENERATING PROCEDURES. THIS FACE SHIELD DOES NOT ELIMINATE THE RISK OF CONTRACTING INFECTION, ILLNESS, DISEASE, OR DEATH, INCLUDING FROM VIRUSES OR OTHER HARMFUL AGENTS. BEFORE USE, THE USER MUST READ AND UNDERSTAND THIS WARNING. ALL APPLICABLE REGULATORY PROVISIONS AND EMPLOYER RESPIRATORY PROTECTION POLICIES MUST BE FOLLOWED. PRODUCT MISUSE MAY ALSO RESULT IN SICKNESS OR DEATH. THIS FACE SHIELD HAS BEEN MANUFACTURED CONSISTENT WITH THE INTENT OF THE CDC GUIDANCE FOR CORONAVIRUS DISEASE 2019 (STRATEGIES FOR OPTIMIZING THE SUPPLY OF FACE SHIELDS).


DISCLAIMER

THIS FACE SHIELD IS NOT AN FDA-APPROVED MASK OR RESPIRATOR, AND EATON DOES NOT WARRANT THAT THIS FACE SHIELD PROVIDES EQUIVALENT PROTECTION. THIS FACE SHIELD HAS NOT BEEN EVALUATED UNDER THE 16 CFR 1610 FLAMMABILITY REQUIREMENTS SO THE USER MUST TAKE ALL NECESSARY PRECAUTIONS; AS SUCH THE PRODUCT SHOULD NOT BE USED IN THE PRESENCE OF ANY HIGH INTENSITY HEAT SOURCE OR FLAMMABLE GAS.

Elastic strap



Head piece (top view)

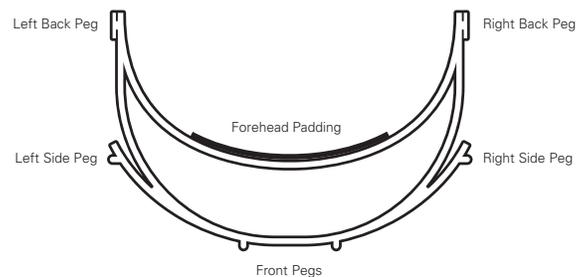


Powering Business Worldwide

Assemble your face shield


- 1 Remove the protective films on plastic face shield by peeling from the edges.

Protective film is present on both sides of the face shield. Either side of the shield can be used as the front.

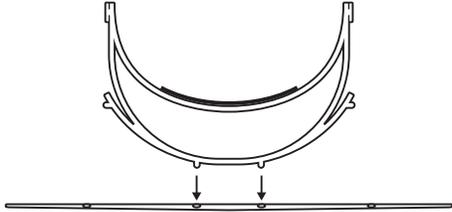


- 2 Insert the front pegs from the head piece into the inner slots on the face shield.

Ensure the Eaton logo is right side up and pegs are on the bottom.

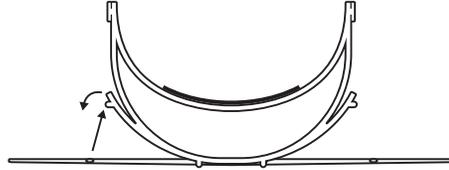


- 3 Bend the side pegs of the head piece outward to insert into the slots on the face shield.

Both pieces are flexible.

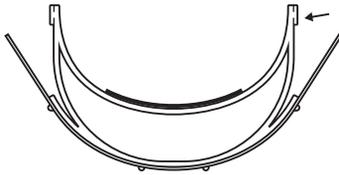


- 4 Slot the elastic strap into place on the back pegs of the head piece.

It is notched and can be adjusted for fit and comfort.

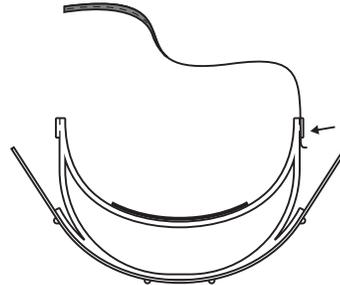

- 5 Once the elastic strap is adjusted comfortably, bend the ends of the face shield to slide the outer notches behind the back pegs of the head piece.

Disassemble by reversing the steps above.

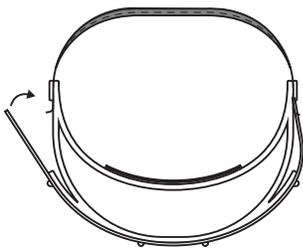

Cleaning your face shield

The face shield can be easily disassembled by reversing the listed assembly steps. You can sterilize this face shield for reuse. Clean each part regularly based on the best practices outlined by your organization or the CDC.

Eaton
1000 Eaton Boulevard
Cleveland, OH 44122
United States
Eaton.com

© 2020 Eaton
All Rights Reserved
Printed in USA
Publication No. IL083001EN / VCG
May 2020